

## PrepCheck: the revolutionary prep-marking system.




PrepCheck promotes adequate reduction allowing ideal thickness of the final restoration and ensures superior strength and aesthetics. PrepCheck's special coating marks the tooth in areas that need to be reduced.

For instructions on use, please see other side ⇨

*Enclosed is a PrepCheck for your next Zirconia case*

**1. Prepare crown prep, available in:**

- 1mm occlusal reduction
- 1.5mm clearance
- 2mm occlusal reduction

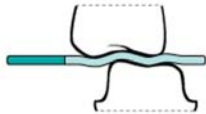

**2. PrepCheck Bite**

Ask patient to bite on the PrepCheck tab. Do NOT pull or tug on the tab. A properly prepared tooth will resist tug even if occlusal reduction is adequate.



**3. Occlusal Check**

Look at occlusal contacts on adjacent teeth. These should appear to occlude when the tab is in place. If the occlusal contacts do not meet, adequate reduction has not been achieved. Remove the tab, look at the preparation and reduce areas that mark.



You can purchase an individual PrepCheck for \$1.75 each, or \$75 for a box (50 in a box).